

# OxyMask™ Quiz



Name: \_\_\_\_\_ Title: \_\_\_\_\_  
Department: \_\_\_\_\_ Employee ID: \_\_\_\_\_  
Employee's Email: \_\_\_\_\_  
Supervisor's Email: \_\_\_\_\_  
Hospital, Clinic, Company \_\_\_\_\_ City: \_\_\_\_\_  
Province / State: \_\_\_\_\_ Country: \_\_\_\_\_

1. What is the FiO<sub>2</sub> delivery capability of the OxyMask™ family? (select one)
  - a) 24% to 90%
  - b) 22% to 65%
  - c) 24% to 90% for all except OxyMask Tyke, which is 22% to 65%
2. What is (are) the patient benefits of the OxyMask™ family? (select one)
  - a) Clear communication while mask is in place
  - b) Reduced risk of claustrophobia and aspiration
  - c) Able to receive fluids without interrupting oxygen therapy
  - d) All of the above
3. Proper placement and fit of the Oxy family of products is **not** important, as they fit and function like any other oxygen mask. (select one)
  - a) True
  - b) False
4. A properly fitted OxyMask™ should? (select one)
  - a) Have the top of mask placed below the level of the eyes
  - b) Rest parallel to the patient's face
  - c) Have the tubing swivel/pin assembly placed just above the upper lip
  - d) All of the above
5. There are 4 sizes of OxyMask™. (select one)
  - a) True
  - b) False
6. What are the clinical advantages of OxyMask™? (select one)
  - a) Oxygen delivery is not interrupted by the need to remove the mask frequently
  - b) FiO<sub>2</sub> titration is simple by adjusting the flow meter up or down
  - c) Access for oral care, oral medication, breathing treatments and the ability to eat and drink.
  - d) All the above
7. How do you titrate O<sub>2</sub> concentration on the OxyMask? (select one)
  - a) Replace the colored Venturi Jets to the specific FiO<sub>2</sub> required
  - b) Adjust Flow Rate up or down to achieve desired oxygen level
  - c) You can't, you must change to another mask