

Sample Oxygen Administration Protocol

Policy and Procedure

	ENTITY/HOSPITAL	NUMBER
	MANUAL Respiratory Care Policy	EFFECTIVE DATE 04-09
	SUBJECT Oxygen Administration Protocol	REVISED 2-11, 3-15, 4-15

1.0 PURPOSE

- 1.1 To establish guidelines for the administration of oxygen at concentrations greater than that in ambient air with the intent of treating or preventing hypoxia.

2.0 POLICY

This oxygen Administration Protocol establishes guidelines for the administration of supplemental oxygen while using **OxyMask**.

This protocol outlines the titration required for patients to maintain targeted SpO₂ on the different devices used.

Oxygen may be titrated from 1 to 6 lpm via nasal cannula, or 1 to 15 lpm via **OxyMask**. If oxygen is titrated above 5 lpm on the **OxyMask** a physician must be notified.

OxyMask