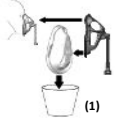
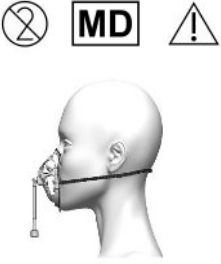
thebetteroxygenmask.com

OT-1025-8 - Oxy₂Mask™ Tyke 7' Tubing

OT-7025-8 - Oxy₂Mask™ Tyke Non PVC 7' Tubing

O ₂ Flow* (LPM)	Approximate** FIO ₂
0.25	22%
0.50	25%
0.75	30%
1	35%
1.50	43%
2	47%
3	50%
≥5	65%

UK CA 8532

Emergo Consulting (UK) Limited

c/o Cr 360—UL International

Compass House, Vision Park Histon

Cambridge, CB24 9BZ

United Kingdom

EC REP

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

CH REP

MedEnvoy Switzerland

Gotthardstrasse 28

6302 Zug

Switzerland

50 Alliance Blvd. Barrie, Ontario
Canada L4M 5K3
705-726-9383
705-728-9537 FAX
thebetteroxygenmask.com

MADE IN CANADA

IFU02871
Issue Date: 09/26/25

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

EN INTENDED USE: A minimal contact, open oxygen delivery device for use with USP medical grade oxygen.

EXPECTED CLINICAL BENEFITS: The Oxy₂Mask™ enables the delivery of supplemental oxygen up to 15+ LPM, targeting an oxygen saturation range in the treatment of hypoxemia and relief of shortness of breath.

CAUTIONS: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. The product is disposable and is not intended for use greater than 29 days. The device has a 5-year shelf life from the date of manufacture. Date of manufacture is stated on labelling. Store device in original packaging until use. Inspect product and packaging prior to use. Do not use if product appears to be damaged.

*Actual flow rate required should be prescribed in conjunction with pulse oximetry.

**Actual FIO₂ will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.

Remove and discard the mask packaging insert (1).

REPORTING SUSPECTED SIDE EFFECTS: If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.

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