thebetteroxygenmask.com

REF

OM-1125-8 - Oxy2Mask™ Adult 7' Tubing
OM-7025-8 - Oxy2Mask™ Adult Non PVC 7' Tubing
OM-1425-8 - Oxy2Mask™ Adult Ear Elastics 7' Tubing
OM-1125-14 - Oxy2Mask™ Adult 14' Tubing
OP-1125-8 - Oxy2Mask™ Plus 7' Tubing
OP-7025-8 - Oxy2Mask™ Plus Non PVC 7' Tubing
OP-1425-8 - Oxy2Mask™ Plus Ear Elastics 7' Tubing
OP-1125-14 - Oxy2Mask™ Plus 14' Tubing
OK-1125-8 - Oxy2Mask™ Kid 7' Tubing
OK-7025-8 - Oxy2Mask™ Kid Non PVC 7' Tubing
OK-1125-14 - Oxy2Mask™ Kid 14' Tubing

O ₂ Flow* (LPM)	Approximate** FIO ₂
1	24% - 27%
2	27% - 32%
3	30% - 60%
4	33% - 65%
5	36% - 69%
7	48% - 80%
10	53% - 85%
12	57% - 89%
≥15	60% - 90%

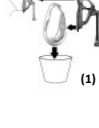
OP-1'-25'-14
 OP-7025-8
 OP-1'-25'-8
 OM-1'-25'-14
 OM-7025-8
 OM-1'-25'-8
 OK-1'-25'-14
 OK-7025-8
 OK-1'-25'-8

OM-1425-8
OP-1425-8

(1)

UK CA 8532
Emergo Consulting (UK) Limited
 c/o Cr 360—UL International
 Compass House, Vision Park Histon
 Cambridge, CB24 9BZ
 United Kingdom

EC REP
EMERGO EUROPE
 Westervoortsedijk 60
 6827 AT Arnhem
 The Netherlands

CH REP
MedEnvoy Switzerland
 Gotthardstrasse 28
 6302 Zug
 Switzerland

CE 2862

Southmedic

50 Alliance Blvd. Barrie, Ontario
 Canada L4M 5K3
 705-726-9383
 705-728-9537 FAX
 thebetteroxygenmask.com

MADE IN CANADA

IFU02881
 Issue Date: 09/26/25

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

EN INTENDED USE: A minimal contact, open oxygen delivery device for use with USP medical grade oxygen.

EXPECTED CLINICAL BENEFITS: The Oxy2Mask™ enables the delivery of supplemental oxygen up to 15+ LPM, targeting an oxygen saturation range in the treatment of hypoxemia and relief of shortness of breath.

CAUTIONS: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. The product is disposable and is not intended for use greater than 29 days. The device has a 5-year shelf life from the date of manufacture. Date of manufacture is stated on labelling. Store device in original packaging until use. Inspect product and packaging prior to use. Do not use if product appears to be damaged.

*Actual flow rate required should be prescribed in conjunction with pulse oximetry.
 **Actual FIO₂ will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.

Remove and discard the mask packaging insert (1).

REPORTING SUSPECTED SIDE EFFECTS: If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.

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