

- •Reliable nasal and oral EtCO2 sampling
 - •Ideal for endoscopic access



Oxy₂Mask™ EtCO₂ Adult


Oxy₂Mask™ EtCO₂ Plus Oxy₂Mask™ EtCO₂ Kid



Flexible oxygen titration from 1 to 15+ lpm

Oxy₂Mask™EtCO₂ is a highly efficient "open" mask system that eliminates the need for a closed design, valves and reservoirs. It features an improved triangular diffuser mushroom disc insert that refines the shape of the oxygen vortices & directs oxygen flow with accelerated velocity toward the patient's nose and mouth.

Delivers oxygen flows from 1-15+ lpm, 24 to 83% FiO₂ while monitoring expired CO₂

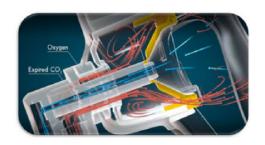

Oxy₂Mask™ EtCO₂Flow Rates
Actual FiO₂ will vary. Breath rate, depth
of breathing and distance of the diffuser
from the face will affect performance.
Actual flow rate required should be
prescribed in conjunction with the use
of a pulse oximeter.

Oxygen Flow Settings	Approx. O ₂ Concentrations
1 LPM	24 - 34
2 LPM	30 - 50
4 LPM	37 - 55
6 LPM	47 - 70
8 LPM	50 - 79
10 LPM	54 - 80
12-15 LPM	59 - 83





Reliable nasal and oral EtCO₂ sampling


Although carbon dioxide escapes through the mask openings during exhalation, it is also sampled through the improved diffuser cup. The cup is attached to a sampling line that can be connected to a sidestream monitor to obtain EtCO₂ readings that are reliable, even at higher O₂ flows.

Detected EtCO₂vs. Time Waveforms using 5% EtCO₂

Reliable EtCO₂ monitoring trends at oxygen flows up to 15 liters per minute

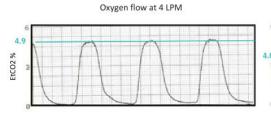

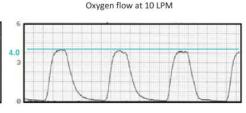

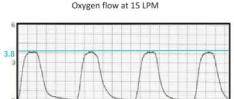

Code	Description	Mask Size	Tubing Length	Elastic
OM-2125-8	OxyMask™ EtCO₂ Adult 7′ oxygen tubing, 8″ pigtail sampling line	Adult	7'	Single
OM-2125-8SLM	OxyMask™ EtCO ₂ Adult 7' oxygen tubing, 8' sampling line with male connector	Adult	7′	Single
OM-2125-14	OxyMask™ EtCO₂ Adult 14' oxygen tubing, 8" pigtail sampling line	Adult	14'	Single
OM-2125-14SLM	OxyMask™ EtCO ₂ Adult 14' oxygen tubing, 15' sampling line with male connector	Adult	14'	Single
OP-2125-8	OxyMask™ EtCO₂ Plus 7' oxygen tubing, 8" pigtail sampling line	Plus	7'	Single
OP-2125-8SLM	OxyMask™ EtCO₂ Plus 7' oxygen tubing, 8' sampling line with male connector	Plus	7'	Single
OP-2125-14	OxyMask™ EtCO₂ Plus 14' oxygen tubing, 8" pigtail sampling line	Plus	14'	Single
OP-2125-14SLM	OxyMask™ EtCO₂ Plus 14' oxygen tubing, 15' sampling line with male connector	Plus	14'	Single
OK-2125-8	OxyMask™ EtCO₂ Kid 7' oxygen tubing, 8" pigtail sampling line	Kid	7′	Single
OK-2125-8SLM	OxyMask™ EtCO₂ Kid 7' oxygen tubing, 8' sampling line with male connector	Kid	7'	Single





Manufactured by: Southmedic Inc. 50 Alliance Blvd., Barrie, ON Canada L4M 5K3 www.thebetteroxygenmask.com email: custserv@southmedic.com 1-800-463-7146 Toll free in North America 1-705-726-9383 1-705-728-9537 Fax

1-705-728-9537 Fax ISO 13485 P0278 Feb 2024

