MD

Rx Only

REF

O ₂ Tube	Sam-pling	Oxy ₂ Mask™ EtCO ₂	Oxy ₂ Mask™ EtCO ₂	Oxy ₂ Mask™ EtCO ₂
7 feet	SLF 8"	OK-2125-8	OM-2125-8	OP-2125-8
	SLM 8'	OK-2125-8SLM	OM-2125-8SLM	OP-2125-8SLM
14 feet	SLF 8"	-	OM-2125-14	OP-2125-14
	SLM 15'	-	OM-2125-14SLM	OP-2125-14SLM

O ₂ Flow* (LPM)	Approximate** FIO ₂
1	24% - 34%
2	30% - 50%
4	37% - 55%
6	47% - 70%
8	50% - 79%
10	54% - 80%
12-15	59% - 83%

UK CA 8532
Emergo Consulting (UK) Limited
c/o Cr 360—UL International
Compass House, Vision Park Histon
Cambridge, CB24 9BZ
United Kingdom

EC REP

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

CH REP
MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

CE 2862

Southmedic
50 Alliance Blvd.
Barrie, Ontario
Canada L4M 5K3
705-726-9383
705-728-9537 FAX
thebetteroxygenmask.com

MADE IN CANADA
IFU0299F
Issue Date: 10/01/25

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

EN INTENDED USE: A minimal contact, open oxygen delivery device for use with USP medical grade oxygen for monitoring breathing by providing a means to sample exhaled CO₂.

EXPECTED CLINICAL BENEFITS: The Oxy-Mask™ O₂/EtCO₂ enables the delivery of supplemental oxygen up to 15+ LPM, targeting an oxygen saturation range in the treatment of hypoxemia and relief of shortness of breath. The device enables continuous EtCO₂ monitoring for nasal and oral breathing which can provide an early warning of potential respiratory conditions.

WARNINGS: The device facilitates EtCO₂ sampling when used in conjunction with capnography equipment. The displayed CO₂ values or any other parameter/index driven from CO₂ will vary with the sampling port position, oxygen flow rate and respiratory pattern.

CAUTIONS: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. The product is disposable and is not intended for use greater than 29 days. The device has a 5-year shelf life from the date of manufacture. Date of manufacture is stated on labelling. Store device in original packaging until use. Inspect product and packaging prior to use. Do not use if product appears to be damaged.

*Actual flow rate required should be prescribed in conjunction with pulse oximetry.

**Actual FIO₂ will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.

Remove and discard the mask packaging insert (1).

REPORTING SUSPECTED SIDE EFFECTS: If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.

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