REF

OMMK-4025-8 - OxyMulti-Mask™ Adult with Nebulizer

OPMK-4025-8 - OxyMulti-Mask™ Plus with Nebulizer

MD

O <sub>2</sub> Flow* (LPM)	Approximate** FIO <sub>2</sub>
1	23% - 26%
2	27% - 33%
3	32% - 48%
4	45% - 57%
6	56% - 68%
8	64% - 74%
10	70% - 77%
12	70% - 81%
≥15	72% - 83%

MADE IN CANADA

IFU01295

Issue Date: 2025/11/13

50 Alliance Blvd. Barrie, Ontario Canada L4M 5K3 705-726-9383 705-728-9537 FAX thebetteroxygenmask.com

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

EN

(1) OXYGEN MASK WITH HIGH HUMIDITY - ADULT

with Standard 22 mm Connector, HSiner Nebulizer #30504 and 7' (2.1 m) Oxygen Tubing

INTENDED USE: A minimal contact, open oxygen delivery device for use with USP medical grade oxygen and delivery of aerosolized drug therapy or high humidity therapy.

EXPECTED CLINICAL BENEFITS: The OxyMulti-Mask™ enables the delivery of supplemental oxygen up to 15+ LPM, targeting an oxygen saturation range in the treatment of hypoxemia and relief of shortness of breath. The device supports the delivery of aerosolized medication and humidity to treat respiratory conditions.

CAUTIONS: Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death. Do not smoke while using this device. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. The product is disposable and is not intended for use greater than 29 days. The device has a 3-year shelf life from the date of manufacture. Date of manufacture is stated on labelling. Store device in original packaging until use. Inspect product and packaging prior to use. Do not use if product appears to be damaged.

\*Actual flow rate required should be prescribed in conjunction with pulse oximetry.

\*\*Actual FIO<sub>2</sub> will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance.

Remove and discard the mask packaging insert (2).

REPORTING SUSPECTED SIDE EFFECTS: If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.

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